

Grace & Holy Trinity Church

an Episcopal parish on Monroe Park



How to Use Assistive Listening

We want you to hear every word of worship.

Our livestream system uses the Williams AV Receiver to connect hearing aids and headphones.

Earbuds or Headphones

For those who want to listen like a radio.



- 1. Plug In:** Put your headphones into the round hole on top.
- 2. Power On:** Press and hold the Power button until the screen lights up.
- 3. Volume:** Press (+) or (-) on the side to find your comfort level.

Neck Loop (T-Coil)

For hearing aids or cochlear implants with a “T” setting.



- 1. Connect:** Plug the neck loop into the hole on top of the receiver. *(Continues)*

- 2. Wear:** Place the loop over your head and around your neck.
- 3. Switch:** Turn your hearing aid to the “T” (Telecoil) position.
- 4. Adjust:** Use the receiver’s (+) and (–) buttons to control the signal.

Auracast – Direct to You

For the newest hearing aids or smartphones with Bluetooth.



Hearing Aids: Your aids may find the “Grace & Holy Trinity” stream automatically.



Smart Phones: Go to your Bluetooth Settings and look for “Broadcast Audio” or “Auracast” to join the church stream directly.

Need Help? Just wave to an Usher!

Please return the receiver after the service.
Thank you!

Grace & Holy Trinity Church
8 N. Laurel Street • Richmond, VA 23220
(804) 359-5628
ghtc.org